



# How to adjust the seat length: GEN\_3




**SHORTEST LENGTH: Less than 41 cm**  
Measuring from their back to the back of their knees

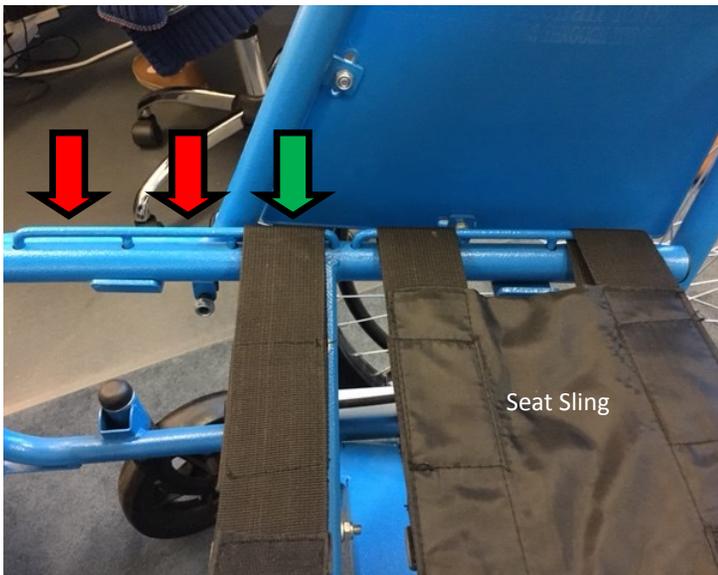

**STEP 1: Move strap closest to the back sling.**  
Remember to tie it underneath

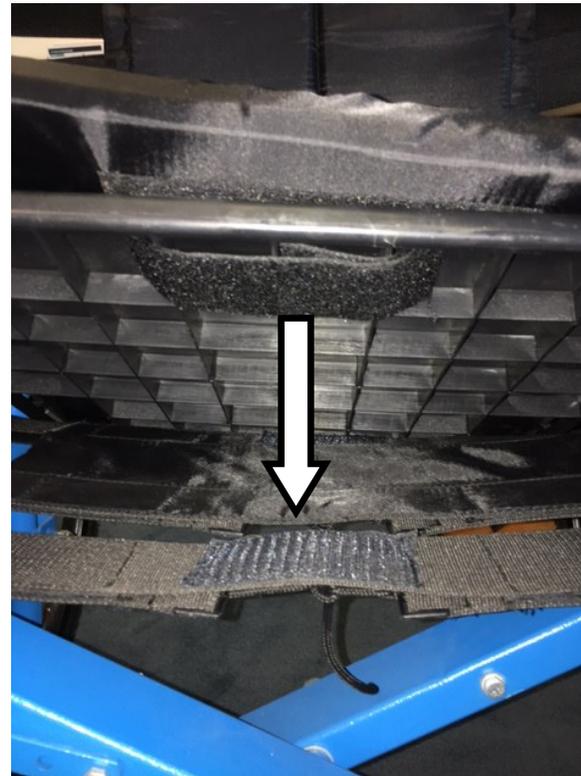

**STEP 2: Line up the Velcro under the front of the hard seat to the Velcro on the strap.**

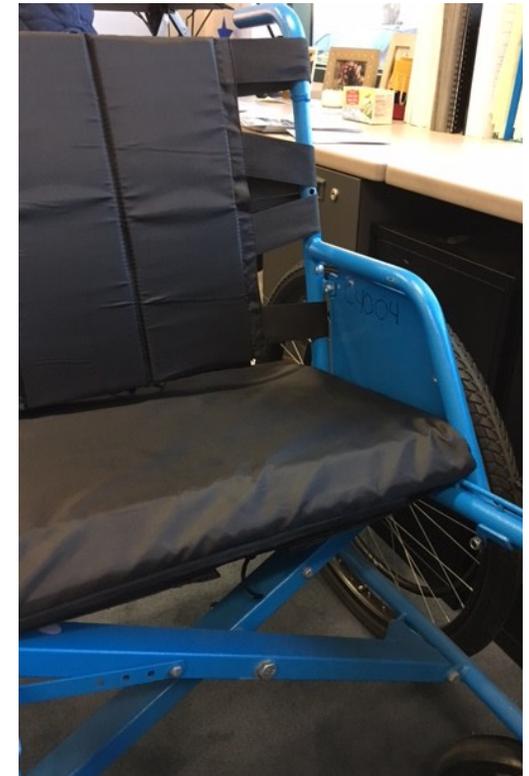


**Finished**

# How to adjust the seat length: GEN\_3




**MIDDLE LENGTH: 41-49 cm**

Measuring from their back to the back of their knees

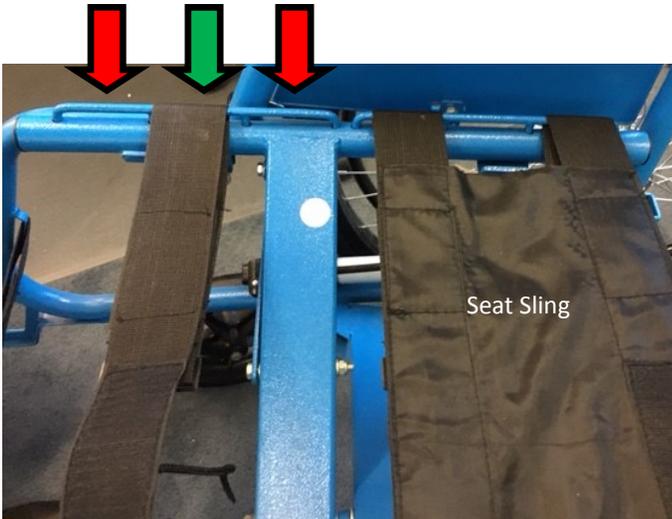

**STEP 1:** Move strap to the middle position on the frame. (see picture)  
Remember to tie it underneath.

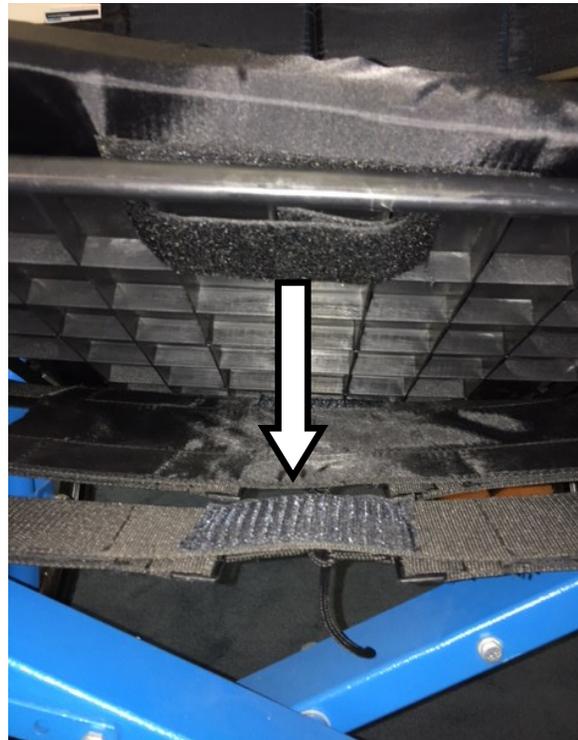

**STEP 2:** Line up the Velcro under the front of the plastic seat to the Velcro on the strap.



**Finished**




## LONGEST LENGTH: Greater than 49 cm

Measuring from their back to the back of their knees



**STEP 1:** Move strap furthest from the back sling.

Remember to tie it underneath.

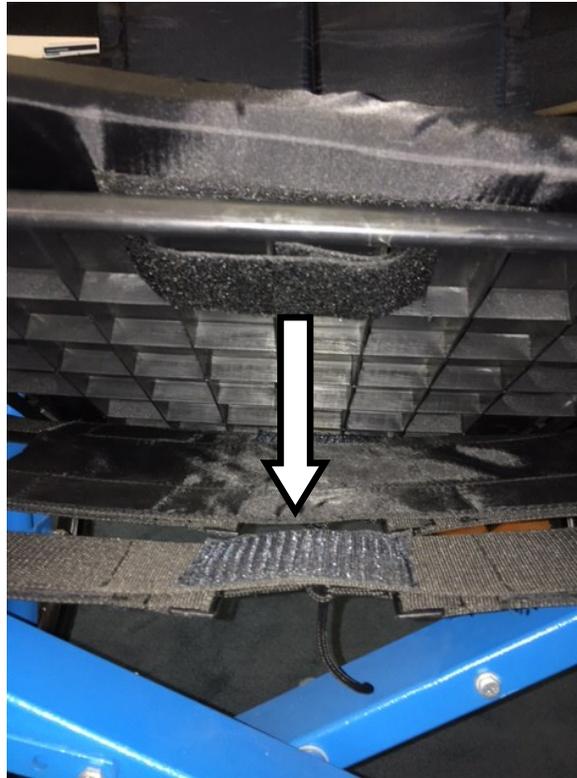

**STEP 2:** Line up the Velcro under the front of the hard seat to the Velcro on the strap.



**Finished.**